Note de service – Memo

2025-10-31

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Note de service – Memo

2025-10-31

To: All adult clinical sectors using BBraun tubing to administer medications via secondary infusion
From: Katherine Mohsen, Nursing Advisor – Vascular Access
Carissa Wong, Nursing Practice Consultant interim
Subject: Malfunctioning of the backcheck valve on the Y-port of the BBraun primary tubing

BBraun has informed us recently again that there is a malfunction of the backcheck valve on the Y-port of the primary infusion tubing where the secondary tubing is attached. This is the same backcheck valve dysfunction that occurred in summer 2024. We ask that you resume the precautions that were implemented in 2024.

Reminder of the issue and actions to take:

The purpose of the backcheck valve is to prevent medication from the secondary tubing from flowing upwards towards the primary bag. As per BBraun, “there is a risk that the intended dose of medication is either not delivered or delivered over an unintentionally extended time frame (simultaneously with the primary infusion)”.

Until further notice, all lots of primary tubing with Y ports are affected. Corrective action is therefore required for all BBraun tubings with Y ports.

Required actions for BBraun volumetric pump users:

- If there is any difficulty priming the tubing, proceed to discard it, complete an incident accident report form (AH-223) and take a new primary tubing.
Reminder: The primary tubing should be primed using the volumetric pump to minimize air bubbles.
- Administer all medications prepared in a syringe by pharmacy using the syringe driver pump (BBraun Perfusor space syringe pump). Do not transfer the medication to an IV bag. When directed by pharmacy to prepare a medication in a syringe, please follow the instructions and administer it using the syringe driver pump.
- Administer all high alert continuous infusion medication as a primary infusion, using a second pump if needed. Do not administer it as a secondary infusion. Consult the list of high alert medication –refer to [Guidelines for Administration of High-Alert Medications – Adult Sites](#).
 - Exceptions: Medications which are pre-spiked on a secondary tubing by pharmacy
- **Practice change:** Administer secondary medications using a volumetric infusion pump. During secondary medication administration, clamp the tubing above the Y port as indicated in the attached procedure. In accordance with the pump reconfiguration done in summer 2024, an alarm will sound at the end of the secondary medication.

Contacts:

- For clinical practice questions, contact katherine.mohsen@muhc.mcgill.ca and / or carissa.wong@muhc.mcgill.ca
- For infusion pump questions, contact anaick.briand@muhc.mcgill.ca
- For product questions, contact purchasing.clinical@muhc.mcgill.ca